

Leg Extensions

Add 2 or 4 inches to your Safety Step. Designed to fit securely inside tubular legs. Powder Coat finish to match your Safety Step. Not Recommended for Folding or Low Commercial Safety Steps.

SSLE-2"

2" Leg Extensions per set

SSLE-4"

4" Leg Extensions per set

Leg Extension Installation

Turn step stool over with platform side down and remove the rubber leg tips.

Insert crimped end of leg extension in the existing leg on the stool. Tap with wood or rubber mallet until snug.

Install screw into existing hole in the leg with screwdriver. If a hole is not present, drill through leg with a 7/64" drill bit, and install screw.

Once the leg extensions are fixed to the legs of the Safety Step Stool, reinstall the leg tips.